Report on field safety corrective action, risk assessment summary and measures for Atlan (batteries), November 2023

Potential Failure Mode

Dräger became aware of a few cases in which the internal backup battery failed spontaneously while the Atlan is being operated without mains supply. This resulted in an unexpected shutdown of the device while it was running on batteries. The affected Atlans started with a battery status of 100%, but quickly shut down and did not trigger the alarm for low battery. A secondary acoustic alarm signal which is independent from mains and battery power was generated as specified. No patient consequences have been reported to Draeger so far.

Root Cause/ Defect or Deficiency

Our analyses indicate that short discharging-/charging cycles, for example caused by briefly disconnecting the device from the mains supply while it is on, might contribute to the observed behavior. The internal battery is designed to back up a loss of mains supply only.

Reported injuries/deaths associated with the problem:

There was no serious injury reported.

Risk Assessment

Potential Effect

If the backup battery fails while the Atlan is being operated without mains supply, the screen goes dark and mechanical ventilation ends. Until the device shuts down the backup battery status may be indicated as 100% and the Atlan might not generate the specified alarms for low battery prior to shutting down. In any case, the Atlan will generate a secondary acoustic alarm signal. It will be necessary to ventilate the patient manually to prevent from serious injury or death.

Severity classification:

Critical (death of one or more person(s))

Probability of an injury

Improbable ($< 1 \times 10^{-4}$)

Risk Classification

With a probability class of “improbable” for a “critical” outcome the residual risk is “considerable”.

Final Risk Statement

The residual risk of a patient injury is considered *not acceptable*, since the observed behavior of a sudden shut-down does not comply with what is specified in the Atlan Instructions for Use and is therefore considered not state-of-the-art.

Corrective Action

All customers of Draeger Atlan anesthesia machines will be informed with a Field Safety Notice about the identified risk.

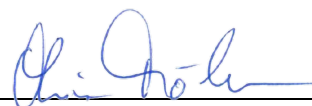
As a second step the Atlan Instructions for Use will be amended with advice how to prevent the observed battery failure and with instructions to perform a regular check of the batteries.

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